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1460 Case Assembly instructions

EN

NOTE: Please read through the entire instruction sheet and determine which configuration you want before beginning any assembly.

- 1** If you want to have the sharps bottle bracket on one end of the case.

Figure 1A

Position the sharps bottle bracket on either end of the case and slide the tabs of the bracket under the lip of the case as shown in Figure 1A:

To attach the bracket, slide the supplied screws through the holes on the case into the threaded inserts in the bracket as shown in Figure 1B:

Hand tighten only, do not over tighten.

Figure 1B

- 2** To install a handle on one or both ends of the case.

Position the handle so that the tines of the handle fit between the ribs on the end of the case as shown in Figure 2A:

Figure 2A

To attach the handle, push the supplied pins smooth end first into the holes as shown in Figure 2B.

Tap the pin in with a hammer until flush and repeat for the other side.

Figure 2B

- 3** To attach the shoulder strap.

- a. To attach the shoulder strap without side handles or sharps bottle bracket, use a handle pin and one ring as shown in figure 3A. Repeat on the opposite side and again on the other end of the case until four rings are installed with four pins.
- b. To attach the shoulder strap with a side handle, follow the steps in 2 above except slide a ring between the outer most case rib and handle tine before fully inserting the pin as shown in Figure 3B and 3C.

Figure 3A

Figure 3B

Figure 3C

Figure 3D

- c. To attach the shoulder strap with the sharps bottle bracket, follow the steps in 1 above except position a ring on the screw between the two outer most ribs before tightening the screw as shown in Figure 3D.

23215 Early Avenue • Torrance, California 90505 USA
Tel 310.326.4700 • FAX 310.326.3311

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